



AlterG® Shorts

The shorts used with the AlterG® Anti-Gravity Treadmill® are available in a wide range of waist sizes (3XS-4XL) to provide a comfortable fit for all body types. Made from high quality neoprene, the shorts will provide excellent performance in both clinical and fitness environments.

Cleaning

For longer life, the shorts should be hand washed with a mild detergent, rinsed and hung dry. They can also be machine washed in cold water on the gentle cycle using standard detergents. All shorts must be air-dried and cannot be placed in the dryer or dry cleaned.

**** Shorts should be disinfected between uses ****

To disinfect shorts, they can be sprayed with a non-toxic commercial grade disinfectant. Successful disinfectant products tested include: *PureGreen24™* & *U-Sol Surface Sanitizer™*. Consult the Center for Disease Control website for the latest guidelines. Use of chlorine bleaches may void warranty. The shorts can tolerate immersion in a 10% non-chlorine bleach solution; however this may lead to discoloration of shorts.

Warranty

Shorts are warranted against manufacturer's defects for 90 days following purchase. If you receive defective shorts, please contact support@alter-g.com or call **(510) 270-5369** to report the problem. As long as your shorts are within the warranty time frame, AlterG will send you a replacement pair at no charge with a pre-paid return-shipping label. In order to avoid being charged, you must ship back your defective shorts to AlterG using the return-shipping label.

Ordering Guidelines

AlterG shorts come in unisex sizes of 3XS – 4XL, and can be purchased for \$75.00 a pair. We also offer a 8-Pack of shorts that includes one of each size for a total of \$515.00. That is a 15% discount!

To order, please visit www.store.alter-g.com For more information please visit us on the web, or contact us via phone at (510) 270-5900.