



DEFY GRAVITY. DEFY LIMITS.

AlterG® Shorts Care

The Anti-Gravity Treadmill® from AlterG can be found in hundreds of Joint Commission Certified facilities across the United States spanning nearly a decade of use. The care and cleaning procedures below have proven effective in care settings in these stringent environments.

The shorts used with the AlterG® Anti-Gravity Treadmill® are available in a wide range of waist sizes (3XS-5XL) to provide a comfortable fit for all body types. Made from high quality neoprene, the shorts will provide excellent performance in both clinical and athletic environments.

Disinfection (*Shorts should be disinfected between uses*)

To disinfect shorts, they can be sprayed with a non-toxic commercial grade disinfectant. Successful disinfectant products tested include: PureGreen24™ & U-Sol Surface Sanitizer™. Consult the Center for Disease Control website for the latest guidelines. Use of chlorine bleaches may void warranty. The shorts can tolerate immersion in a 10% nonchlorine bleach solution; however, this may lead to discoloration of shorts.

Cleaning

For longer life, the shorts should be hand washed with a mild detergent, rinsed and hung dry. They can also be machine washed in cold water on the gentle cycle using standard detergents. All shorts must be air-dried and cannot be placed in the dryer or dry cleaned.

Warranty

Shorts are warranted against manufacturer's defects for 90 days following purchase. If you receive defective shorts, please contact support@alterg.com or call +1.510.270.5900 to report the problem. As long as your shorts are within the warranty time frame, AlterG will send you a replacement pair at no charge with a pre-paid return shipping label. To avoid being charged, you must ship back your defective shorts to AlterG using the return-shipping label.

To order, please visit www.store.alterg.com.

For more information please visit us at alterg.com or call us at +1.510.270.5900.